



CENTRIFUGE CLEANING & DISINFECTION

To prolong the life of the centrifuge, cleaning and disinfection is recommended every six months or whenever there is a spillage or tube breakage. Contaminants must be removed immediately, or corrosion and premature degradation of components can occur. Before using any cleaning or decontamination methods other than those recommended by the manufacturer, users should verify with the manufacturer that the proposed method will not damage the equipment.

- Unplug the centrifuge before cleaning.
- Use appropriate personal protective equipment (PPE).
- Apply cleaning solutions with a dampened towel or cloth ONLY. Do not spray or pour cleaning solution directly onto or into the centrifuge. Do not saturate the centrifuge or submerge the centrifuge in water or other cleaning solutions as this will cause damage, create a safety risk, and void the warranty.
- ONLY isopropyl alcohol or a 10% (5500 PPM) bleach solution should be used to disinfect the centrifuge and its accessories.
- All surfaces must be dried immediately after cleaning and disinfecting.
- Fully/partially halogenated hydrocarbons, ketones, esters, ethers, benzyls, ethyl benzenes, and all other chemicals not prescribed by the manufacturer shall not be used as they may cause damage to the rotor chamber, rotor, carriers, accessories, and centrifuge exterior and void the warranty.



TBQ Germicidal products are not recommended as they may cause damage to the centrifuge. Refrain from using to prevent voiding the warranty.

4721 Emperor Blvd, Suite 100, Durham, NC 27703
T: 1-800-836-4080 E: CustomerServiceUSA@BioventusGlobal.com W: Bioventus.com

Document Detail

Type: Labeling

Document No.: LBL-000788[B]

Title: XCELL PRP Approved Cleaning and Disinfecting Products

Owner: TRAPPER.PRESSLER Trapper Pressler

Status: RELEASED

Review

Level	Owner Role	Actor	Sign-off Date	Sign-off By
0	BV Configuration Analyst BV Configuration Analyst	AMBER.PLOTNER Amber Plotner	29-Jul-2025 3:10 pm	AMBER.PLOTNER
1	BV Doc Owner / Author BV Doc Owner / Author	TRAPPER.PRESSLER Trapper Pressler	29-Jul-2025 6:43 pm	TRAPPER.PRESSLER
1	BV Doc Approver BV Doc Approver	LIZZIE.FIELDS Lizzie Fields	29-Jul-2025 8:11 pm	LIZZIE.FIELDS
1	BV Doc Approver BV Doc Approver	STEPHANIE.PHIPPS Stephanie Phipps	29-Jul-2025 7:02 pm	STEPHANIE.PHIPPS
1	BV Doc Approver BV Doc Approver	BRETT.OWEN Brett Owen	29-Jul-2025 4:35 pm	BRETT.OWEN
1	BV Doc Approver BV Doc Approver	RAY.PERRY Ray Perry	29-Jul-2025 7:08 pm	RAY.PERRY
2	BV Configuration Analyst BV Configuration Analyst	AMBER.PLOTNER Amber Plotner	29-Jul-2025 8:44 pm	AMBER.PLOTNER